


fasciotens®Abdomen

# Quick Guide




**Tip:**  
Leave the Quick Guide  
near the patient so the  
treatment team can ac-  
cess it for information.

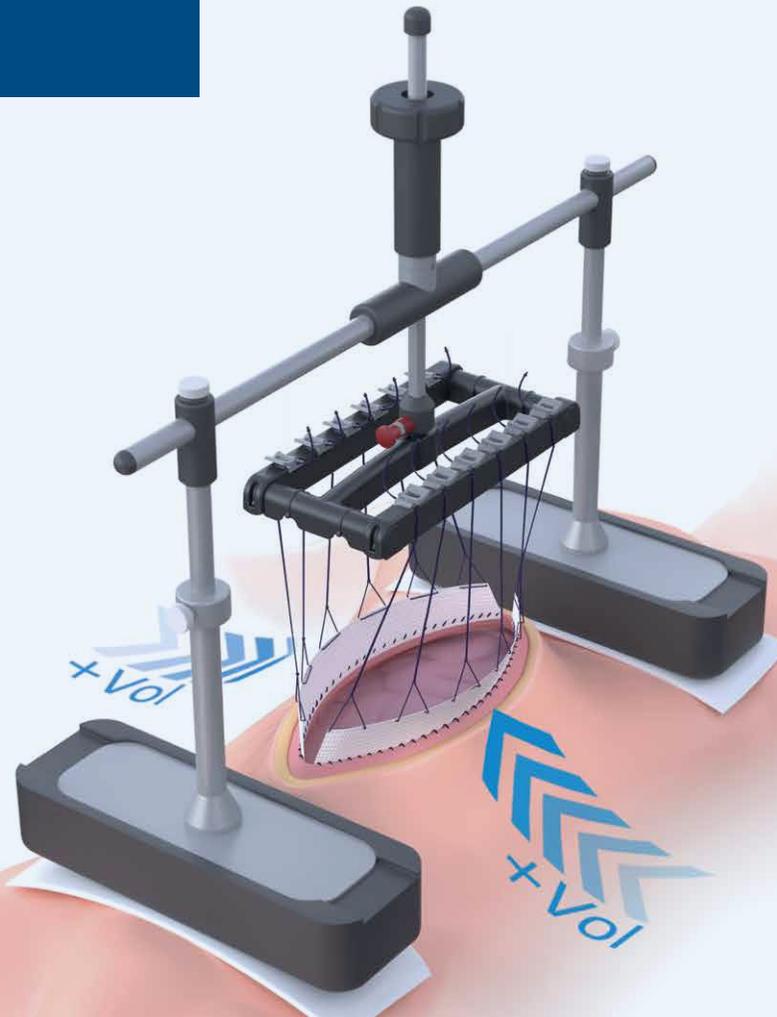

**fasciotens**  
ABDOMINAL WALL SOLUTIONS

## Product design

▲ At the beginning of each new assembly, spin the black screw head upwards until it is flush with the end cap.

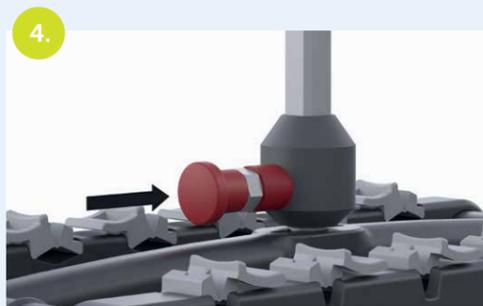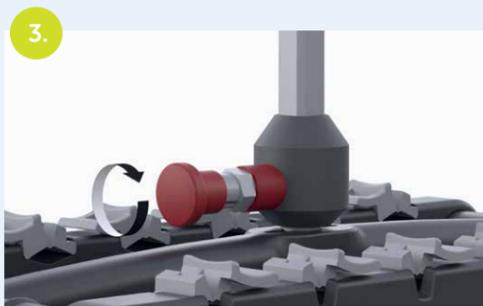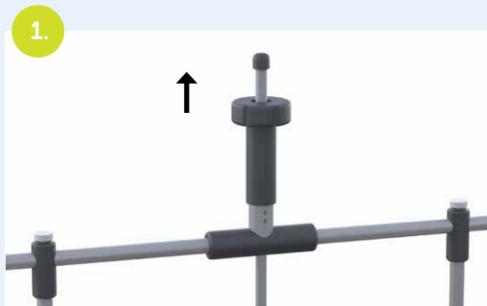

## Height and length adjustment (optional)






## Tensile force adjustment







- ▲ Turn the black screw head to adjust the tensile force.
- ▲ Check the tensile force regularly and, if necessary, adjust it to the previously defined range.

## Tensioning the sutures

▲ The basic tension of all sutures must be checked. If necessary, retighten the sutures.

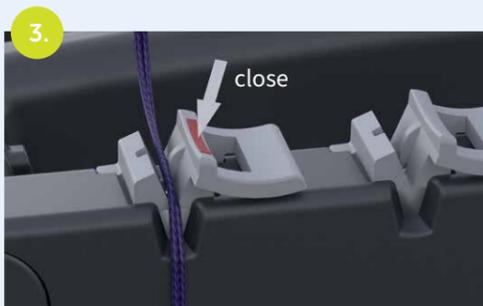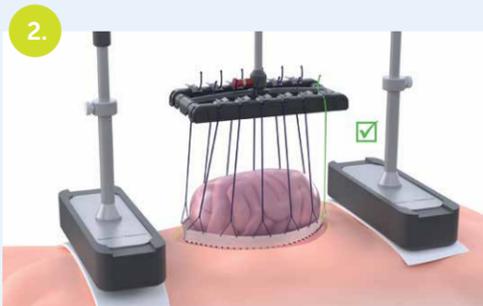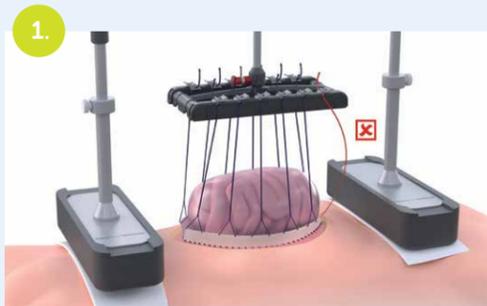

## Disassembly in daily care (and emergencies)

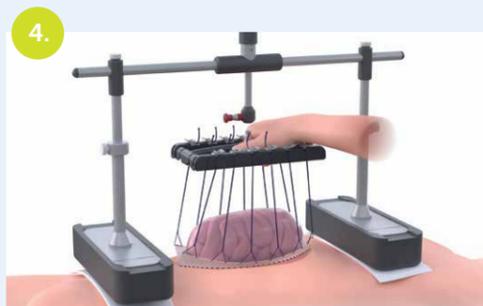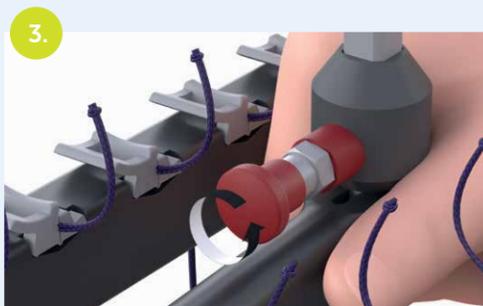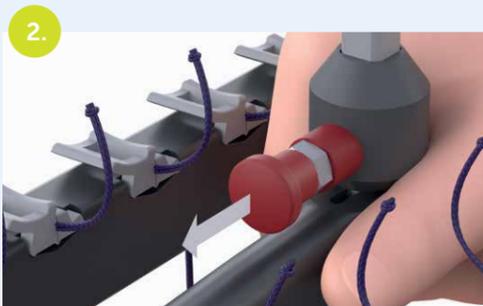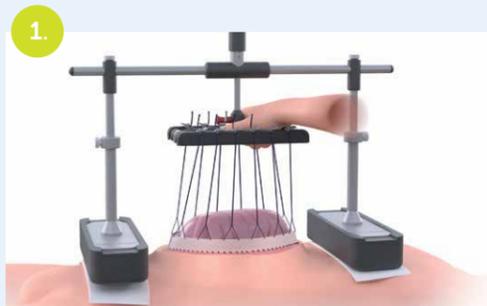

Hold the suture retainer with one hand.

Retract the red locking bolt.

Rotate it by 90°.

The suture retainer is now detached and can be removed by pulling downwards.

## ▲ Further safety instructions

The support surfaces may only be deployed on intact skin.

Ensure that no other objects e.g. cables, electrodes, feed or discharge lines are present under the support surfaces.

Always cushion the support surfaces with large crease-free absorbent pads or similar crease-free materials.

The product must not be placed on the genitals.

To avoid skin irritation, traction intervals of about 5 hours followed by a one-hour break are recommended.

Video guide



**fasciotens**  
ABDOMINAL WALL SOLUTIONS

Please note the instructions for use!

# fasciotens

ABDOMINAL WALL SOLUTIONS



**fasciotens GmbH**

Moltkeplatz 1  
D-45138 Essen | Germany

**www.fasciotens.de**

info@fasciotens.de  
+49 (0)221 17738500

**Support:**

support@fasciotens.de  
Tel.: +49 (0)221 17738500

**fasciotens GmbH**

Aachener Str. 1053-1055  
D-50858 Cologne | Germany